


Disposable

Circumcision Stapler

Pain reducing; Speed up the healing process;
Neat incision.

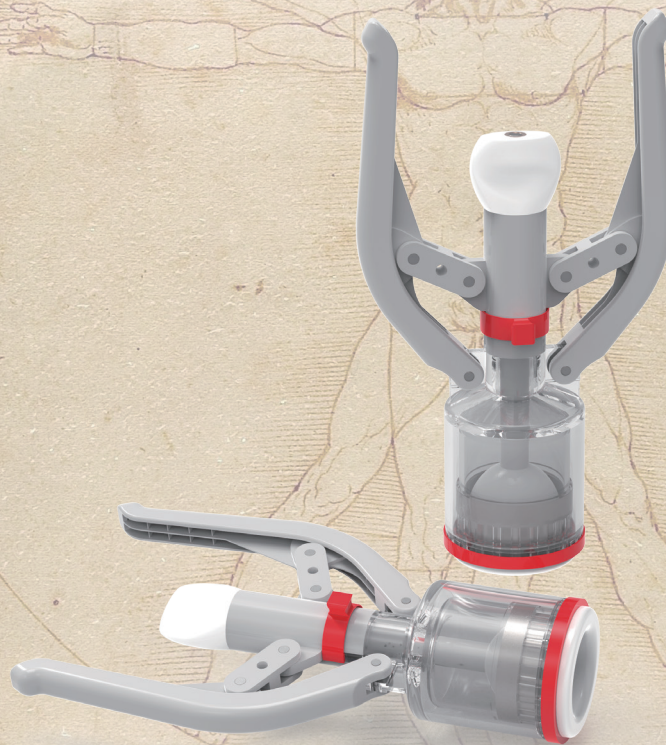

Disposable Circumcision Stapler

Manufacturer: Weihai Medison Medical Equipment Co.,Ltd
The Registered Address: Qujiahe Industrial Park, Yangting Town,
Huancui District, Weihai City, Shandong Province, China

Medical Equipment Registration Number:
Lu Machinery Registration Approval 20192020192

Product Name

Disposable Circumcision Stapler

Specification

MDCS-A-12、MDCS-A-16、MDCS-A-22、MDCS-A-26、MDCS-A-30、MDCS-A-36

Product Feature

- Less pain. Smooth incision edges. Faster recovery.
- Save time for sutures& hemostasis.
- Due to the sutureless ligation during the operation, the foreign body discomfort caused by the residual thread after ligation and hemostasis is avoided.



Advantages compare to existing surgery:

- The foreskin edge is neat and symmetrical, and the appearance is more beautiful.
- The patient's postoperative comfort is significantly improved.
- In routine surgery, doctors are easy to hurt themselves and cause wound infection. The circumcision stapler can prevent accidents to the greatest extent.

Applicable person, scope and departments:

Applicable person: It is suitable for children and adults with long foreskin or phimosis, regardless of age.
Scope of application: clinical circumcision and suture.
Applicable Department: common operating room of Urology.

Contraindication:

Foreskin and glans deformity (including extensive and close adhesion of the foreskin and glans), foreskin and glans infection, edema, suspected penile cancer, suffering from systemic diseases (such as bleeding tendency, hypoalbuminemia, severe cardiovascular disease, etc.)

Instructions for use

01

Use hemostatic forceps to clamp the ventral side of the foreskin (at the frenum) at the transition of the inner and outer plates.
Use two to four hemostatic forceps to clamp other parts in order to lift the foreskin;

02

Secure the foreskin to the mover with a zip tie or silk thread.
Put the instrument on, screw on and tighten the positioning adjustment nut to the end face of the pull rod, which is flush with the back of the knob or slightly protrudes.

03

Take out the safety buckle and press the handle, the foreskin is cut and sutured at the same time.
Release the handle after 30 seconds. When the handle will not release by itself, it can be released by hand. When adjustment the nut, rotate it at 5-10mm.

04

The mover is pushed forward to loosen the cut part and observe whether it is completely cut. If there is any uncut, it can be trimmed with a blade or scissors.
After cutting, directly use elastic bandage for compression dressing. After 30 minutes, loosen the elastic bandage, and cut the silicone rubber ring at the place without blood vessels.